



CPS Aim® SL slittable inner catheter with integrated valve has a low slit force to reduce the risk of lead movement when slitting the catheter.



CPS Direct® SL II slittable outer guide catheter has a slittable hub and integrated shaft to provide a smooth transition when slitting the catheter.

St. Jude Medical is focused on reducing risk by continuously finding ways to put more control into the hands of those who save and enhance lives.

ATRIAL FIBRILLATION  
 CARDIAC RHYTHM MANAGEMENT  
 CARDIOVASCULAR  
 NEUROMODULATION

**Global Headquarters**  
 One St. Jude Medical Drive  
 St. Paul, Minnesota 55117  
 USA  
 +1 651 756 2000  
 +1 651 756 3301 Fax

**Cardiac Rhythm Management Division**  
 15900 Valley View Court  
 Sylmar, California 91342  
 USA  
 +1 818 362 6822  
 +1 818 364 5814 Fax

**St. Jude Medical AB**  
 Veddestavägen 19  
 175 84 Järfälla  
 Sweden  
 +46 8 474 40 00  
 +46 8 760 95 42 Fax

**U.S. Division**  
 807 Las Cimas Parkway  
 Suite 400  
 Austin, Texas 78746  
 USA  
 +1 512 732 7400  
 +1 512 732 2418 Fax

[SJMprofessional.com](http://SJMprofessional.com)



**CAUTION:** FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE, DISTRIBUTION AND USE BY OR ON THE ORDER OF A PHYSICIAN.

**Brief Summary:** Prior to using these devices, please review the User's Manual for a complete listing of indications, contraindications, warnings, precautions, potential adverse events and directions for use. Unless otherwise noted, ® or ™ indicates a registered or unregistered trademark or service mark owned by, or licensed to, St. Jude Medical, Inc. or one of its subsidiaries. ST. JUDE MEDICAL, the nine-squares symbol and MORE CONTROL. LESS RISK. are registered and unregistered trademarks and service marks of St. Jude Medical, Inc. and its related companies. ©2010 St. Jude Medical, Inc. All rights reserved.  
 Item NO. N0972

How to use the  
 CPS® Universal Slitter



The ergonomic design of the CPS® Universal Slitter includes a rubber thumb grip for better stability while slitting St. Jude Medical® catheters. The CPS Universal Slitter is compatible with all St. Jude Medical leads (4 F to 6 F) and all slittable (SL) catheters (CPS Aim® SL, CPS Direct® SL and CPS Direct® SL II).

## USING THE CPS UNIVERSAL SLITTER

### 1. Secure lead in slitter

- Grasp the CPS Universal Slitter in your working hand and position your thumb so you can place it firmly on top of the rubber thumb grip of the lead channel.
- With your other hand, snap the lead into the top plastic groove of the lead channel.
- Verify that the lead fits securely in the lead channel.

### 2. Stabilize lead and position slitter

Firmly hold the lead with your thumb on the rubber thumb grip and use the reference line guide to ensure the slitter remains parallel to the catheter shaft.

### 3. Slit Catheter

Hold the slitter in a stable, fixed position and in one continuous motion, steadily pull the catheter toward you and through the slitter.

Note: It is important to ensure that you hold the slitter parallel to the catheter shaft during the entire slitting motion.

Once the lead is visible and the catheters are removed, carefully remove the lead from the slitter.

